college park
TECHNOLOGY for the HUMAN RACE

technical instructions

التعليمات الفنية • Teknisk vejledning • Technische Anleitung
Τεχνικές οδηγίες • Instrucciones técnicas • Tekniset ohjeet
Instructions techniques • הוראות טכניות • Istruzioni tecniche
Technische instructies • Tekniske instruksjoner • Instrukcje techniczne
Instruções técnicas • Instruções técnicas • Технические инструкции •
Technické pokyny • Tekniska anvisningar • Teknik Talimatlar • 技术说明

FIGURE 1


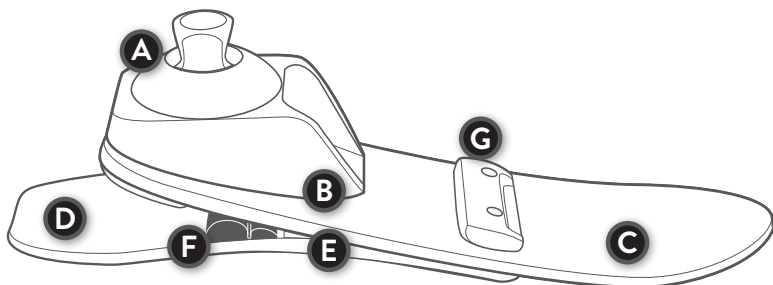
FIGURE 2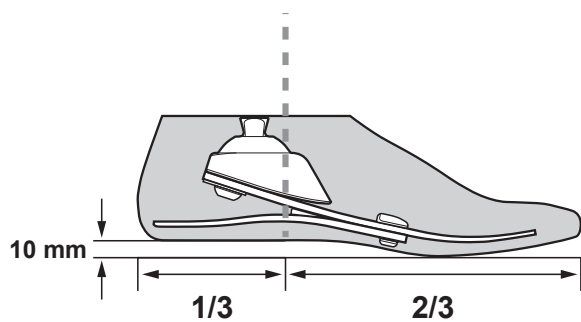

Package Contents

- (1) Horizon HD Foot (1) Foot Shell
- (1) CPI Sock (1) Horizon Wedge Kit
- (1) Cosmetic Attachment Plate (CAP)

Tools Recommended

- (1) 4mm Hex Key
- (1) Foot Horn

This diagram is to help familiarize you with the unique parts of the Horizon HD. These parts are referenced in the instructions and used when speaking with a technical service representative.

Key Components (Figure 1)

- A. Integrated Pyramid B. Housing C. Toe spring D. Heel Spring
- E. Permanent Wedge F. Rounded Heel G. Nut Plate
- Wedge (Optional)
- CPI Sock (not shown) • Foot Shell (not shown)

PRODUCT DESCRIPTION

This prosthetic foot device is constructed with an integrated pyramid, housing, two composite springs, and heel wedge. The toe spring is secured to the housing and heel spring with fasteners.

INTENDED USE

The Horizon HD is a prosthetic foot designed to replace one or more functions of the biologic human foot.

⚠ INDICATIONS:

- Lower limb amputations

⚠ CONTRAINDICATIONS:

- None known

⚠ PROTECTIVE COVER ON DOME

Remove the protective cover on dome after alignment is completed and before patient leaves clinic.

TECHNICAL SPECIFICATIONS

FOOT SIZE	WEIGHT LIMIT	BUILD HEIGHT	FOOT WEIGHT*
21-25 cm	330 lbs / 150 kg	2.3 in / 5.8 cm	559 g
26-30 cm	365 lbs / 165 kg		

*26cm foot w/shell

GAIT MATCHING® GUIDELINES

The gait match determines the firmness of the foot based on the user's specifications (foot size, patient weight, and impact level).

FIRMNESS CATEGORIES

Refer to the chart below to determine the correct firmness category.

Note: Incorrect category selection may result in poor device function. Contact College Park Technical Service if you have questions about category selection.

WEIGHT LBS	0-140	141-180	181-220	221-275	276-330	331-365
WEIGHT KG	0-63	64-81	82-100	101-125	126-150	151-166
SIZE CM	21-30					26-30
LOW-MODERATE IMPACT	1	2	3	4	5	6
HIGH IMPACT	2	3	4	5	6	6

MOUNTING

Use only high quality proximal endoskeletal components.

ASSEMBLY AND DISASSEMBLY (for sock replacement)

Use the Foot Horn to don and doff the foot shell. Remove the CPI Sock and replace as needed. Any further disassembly or modification of components will void the warranty.

STATIC ALIGNMENT (Figure 2)

For optimal function, balance the patient's weight evenly between the heel and toe.

- The Horizon HD was designed with a 3/8" (10 mm) heel rise.
- The load line divides the foot at 1/3 heel lever and 2/3 toe lever.

DYNAMIC ADJUSTMENTS

DESIRED RESULT	ALIGNMENT CHANGE	COMPONENT CHANGE
Firmer Toe Response	Plantarflex the Horizon or move load line posterior	None
Softer Toe Response	Dorsiflex the Horizon or move load line anterior	None
Firmer Heel Response	Dorsiflex the Horizon or move load line anterior	Install Rounded Heel Wedge*
Softer Heel Response	Plantarflex the Horizon or move load line posterior	Remove Rounded Heel Wedge*

*To install or remove toe and heel wedges, refer to the Horizon Wedge Kit Instructions.

WARNING

- Do not expose this product to corrosive materials, salt water or pH extremes.
- Failure to follow these technical instructions or use of this product outside the scope of its Limited Warranty may result in injury to the patient or damage to the product.

RESIDUAL RISK STATEMENT

NOTICE OF RESIDUAL RISK

During fitting process, ensure that CPI sock does not become pinched between foot and endoskeletal componentry.

WARRANTY INSPECTION /MAINTENANCE INFORMATION

High patient weight and/or impact level may require more frequent inspections. We recommend you visually inspect the following applicable parts for excessive wear and fatigue at each warranty inspection.

- Composites and Adapters
- CPI Sock
- Foot Shell
- Heel Wedge (if applied)

WARRANTY INSPECTION SCHEDULE FOR HORIZON HD: ANNUALLY.

TECHNICAL ASSISTANCE / EMERGENCY SERVICE 24-7-365

College Park's regular office hours are Monday through Friday, 8:30 am – 5:30 pm (EST). After hours, an emergency Technical Service number is available to contact a College Park representative.

LIABILITY

The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer

CAUTION

College Park products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standards are achieved only when College Park products are used with other recommended College Park components. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients.

CAUTION

If any problems occur with the use of this product, immediately contact your medical professional. The prosthetist and/or patient should report any serious incident* that has occurred in relation to the device to College Park Industries, Inc. and the competent authority of the Member State in which the prosthetist and/or patient is established.

*'Serious incident' is defined as any incident that directly or indirectly led, may have led, or might lead to any of the following; (a) the death of a patient, user, or other person, (b) the temporary or permanent serious deterioration of a patient's, user's, or other person's state of health, (c) a serious public health threat.

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©2022 College Park Industries, Inc. All rights reserved.