college park
TECHNOLOGY *for the* HUMAN RACE

technical instructions

التعليمات الفنية • Teknisk vejledning • Technische Anleitung
Τεχνικές οδηγίες • Instrucciones técnicas • Teknisket ohjeet
Instructions techniques • הוראות טכניות • Istruzioni tecniche
Technische instructies • Tekniskie instruksjoner • Instrukcje techniczne
Instruções técnicas • Instruções técnicas • Технические инструкции •
Technické pokyny • Tekniska anvisningar • Teknik Talimatlar • 技术说明

FIGURE 1


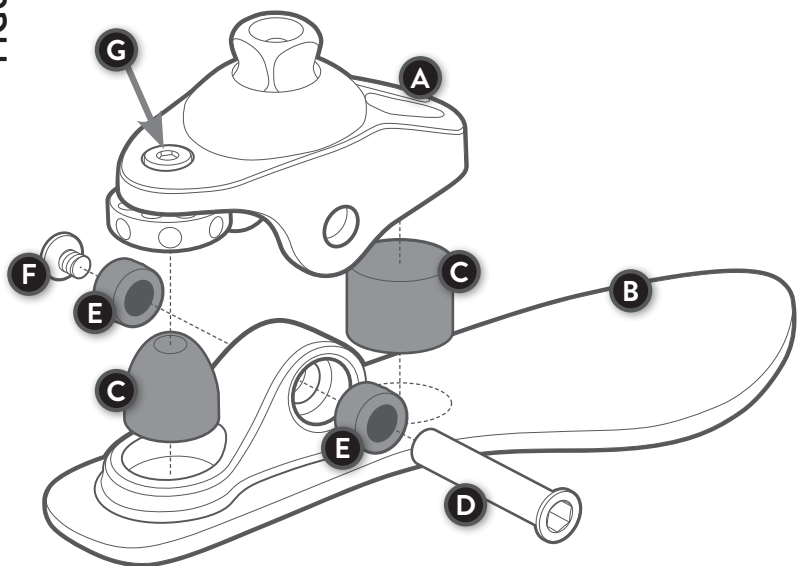
FIGURE 2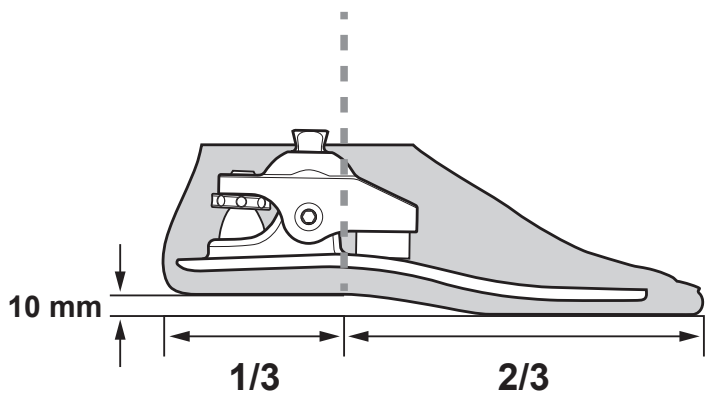

Package Contents

- (1) Tribute Foot (1) Foot Shell
(1) CPI Sock (1) Cosmetic Attachment Plate (*regular toe only*)
(1) TruLube Lubricant

Tools Recommended

- (1) 4 mm Hex Key (1) Foot Horn
(1) 6 mm Hex Key (1) Pin Guide

This diagram is to help familiarize you with the unique parts of the Tribute. These parts are referenced in the instructions and used when speaking with a technical service representative.

Key Components (Figure 1)

- A. Endo Ankle Bone B. Foreheel Bone C. Bumpers (2) D. Axial Pin
E. Ankle Bushings (2) F. Axial Pin Screw Torque 4 N-m (36 in-lbs) G. Stride Control® Adjustment
• CPI Sock (not shown) • Foot Shell (not shown)

PRODUCT DESCRIPTION

This prosthetic foot device is constructed with an endo ankle bone, composite foreheel, bumpers, bushings, axial pin, and screw. The foreheel is secured to the ankle bone with the axial pin and screw.

INTENDED USE

The Tribute is a prosthetic foot designed to replace one or more functions of the biologic human foot.

INDICATIONS:

- Lower limb amputations

CONTRAINDICATIONS:

- None known

PROTECTIVE COVER ON DOME

Remove the protective cover on dome after alignment is completed and before patient leaves clinic.

TECHNICAL SPECIFICATIONS

FOOT SIZE	WEIGHT LIMIT	BUILD HEIGHT	FOOT WEIGHT*
21-30 cm	220 lbs / 100 kg	2.4 in / 6.1 cm	529 g

*26cm foot w/shell

GAIT MATCHING® GUIDELINES

The gait match determines the firmness of the foot based on the user's specifications (foot size, patient weight, and impact level).

FIRMNESS CATEGORIES

Refer to the chart below to determine the correct firmness category.

Note: Incorrect category selection may result in poor device function. Contact College Park Technical Service if you have questions about category selection.

WEIGHT LBS	0-100	101-120	121-140	141-160	161-180	181-200	201-220
WEIGHT KG	0-45	46-54	55-63	64-72	73-81	82-91	92-100
SIZE CM	21-30						
Low Impact	1	1	2	2	3	3	4
Moderate Impact	1	2	2	2	3	4	4

GAIT MATCH CHART

Components may be changed as needed. Determine the user's firmness category, then refer to the chart below for the recommended components.

For gait match adjustments, see section: Dynamic Adjustments.

COMPONENT	FIRMNESS CATEGORY			
	1	2	3	4
FRONT BUMPER*	S	M	M	F
ANKLE BUSHINGS*	M	M	M	F
REAR BUMPER	S	M	F	XF

*Color Code: (S) Soft = Yellow, (M) Medium = Red, (F) Firm = Blue

MOUNTING

Use only high quality proximal endoskeletal components.

ASSEMBLY AND DISASSEMBLY

1. Use the Foot Horn to don and doff the foot shell. Remove the CPI Sock and replace as needed.
2. Use a 4 mm hex key to reduce the preload on the bumpers by turning the Stride Control adjuster counter-clockwise. Record the number of turns for use in re-assembly.
3. Use a 6 mm and 4 mm hex keys to remove the axial pin screw.
4. Attach the pin guide to the axial pin. Push and rotate clockwise.
5. Unscrew the axial pin from the pin guide and remove it from the ankle bone.
6. Remove the ankle bone from the foreheel to access the front and rear bumpers and ankle bushings.
7. For re-assembly, lubricate the axial pin, inside and outside of the ankle bushings and ankle bushing pocket, then reverse steps 1-4. Torque axial pin screw to 4 N·m (36 in·lbs).

DO NOT LUBRICATE FRONT AND REAR BUMPERS.

STATIC ALIGNMENT (Figure 2)

For optimal function, balance the patient's weight evenly between the heel and toe.

- The Tribute was designed with a 3/8" (10 mm) heel rise.
- The load line divides the foot at 1/3 heel lever and 2/3 toe lever.

STRIDE CONTROL ADJUSTMENT

The adjustable Stride Control gives you the ability to customize an individual's gait timing, fine-tuning the foot's response by either turning it clockwise/ counterclockwise. This adjustment affects both plantar and dorsiflexion resistance without the need to change soft components.

The Tribute ships with a medium Stride Control setting. If new bumpers are fitted, set the preload to 2-turns (medium preload) from the point of initial contact with the ankle bone.

If an additional 1-1/2 turns are required to attain the desired resistance, then a change in bumpers is recommended.

DYNAMIC ADJUSTMENTS

DESIRED RESULT	ALIGNMENT CHANGE	COMPONENT CHANGE
Firmer Toe Response	Plantarflex the Tribute or move load line posterior	Increase Stride Control Front Bumper up one step firmer
Softer Toe Response	Dorsiflex the Tribute or move load line anterior	Decrease Stride Control Front Bumper down one step softer
Firmer Heel Response	Dorsiflex the Tribute or move load line anterior	Increase Stride Control Rear Bumper up one step firmer
Softer Heel Response	Plantarflex the Tribute or move load line posterior	Decrease Stride Control Rear Bumper down one step softer

WARNING

- Do not expose this product to corrosive materials, salt water or pH extremes.
- Failure to follow these technical instructions or use of this product outside the scope of its Limited Warranty may result in injury to the patient or damage to the product.
- Any further disassembly or modification of components will void the warranty.

RESIDUAL RISK STATEMENT

NOTICE OF RESIDUAL RISK

During fitting process, ensure that CPI sock does not become pinched between foot and endoskeletal componentry.

WARRANTY INSPECTION / MAINTENANCE INFORMATION

College Park recommends that you schedule your patients for check-ups on the Tribute foot six months after initial fitting and then annually.

High patient weight and/or impact level may require more frequent inspections. Soft component wear depends on the patient weight, impact level and environment. We recommend you inspect the following applicable parts for excessive wear and fatigue at each warranty inspection and replace as needed.

- Soft components (disassemble, inspect & re-lubricate)
- CPI Sock
- Composites and Adapters
- Foot Shell

WARRANTY INSPECTION SCHEDULE FOR TRIBUTE: 6 MONTHS, THEN ANNUALLY.

TECHNICAL ASSISTANCE / EMERGENCY SERVICE 24-7-365

College Park's regular office hours are Monday through Friday, 8:30 am – 5:30 pm (EST). After hours, an emergency Technical Service number is available to contact a College Park representative.

LIABILITY

The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

CAUTION

College Park products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standards are achieved only when College Park products are used with other recommended College Park components. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients.

CAUTION

If any problems occur with the use of this product, immediately contact your medical professional. The prosthetist and/or patient should report any serious incident* that has occurred in relation to the device to College Park Industries, Inc. and the competent authority of the Member State in which the prosthetist and/or patient is established.

*"Serious incident" is defined as any incident that directly or indirectly led, may have led, or might lead to any of the following; (a) the death of a patient, user, or other person, (b) the temporary or permanent serious deterioration of a patient's, user's, or other person's state of health, (c) a serious public health threat.

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©2022 College Park Industries, Inc. All rights reserved.