



THUASNE

GenuStart ROM

Hinged ligament knee brace
with flexion / extension control



GenuStart ROM

Hinged ligament knee brace
with flexion / extension control



TARGET PATIENT

The brace is designed for patients with moderate to severe ligament injuries, or for post-operative treatment.

MODE OF ACTION

Knee support

- Medial and lateral reinforcements made of aluminum and anatomically shaped

Range of motion control

- Setting at 0° extension integrated
- Key flexion and extension limitations


Suspension on the leg

- Intimate fit with proper replication of the knee joint natural movement thanks to the TM5+ hinge
- Prevention of rotation with alternating straps location

EASE OF USE

Angulation stops are easy to set

- Pins at key limitations
- Tool-less hinge cap design
- 90° flexion pin already inserted

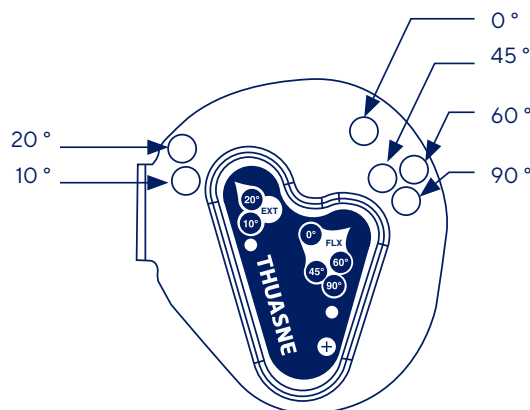

Easy fitting thanks to the asymmetrical anterior closure (open version)

Easy to tighten thanks to the alternating strap location

Easy to adapt to different morphologies thanks to a large sizing range

COMFORT

Patella buttress for support

Light and permeable spacer fabric with a thin elastic knit at the back


Homogeneous closure avoiding tourniquet effect thanks to the half straps

GenuStart ROM

SIZES AND ARTICLES NUMBERS

- A : Thigh circumference 15 cm above knee centre**
B : Knee circumference
C : Calf circumference 15 cm under knee centre

	A	B	C
XS	33 - 39 cm	30 - 33 cm	25 - 30 cm
S	39 - 37 cm	33 - 36 cm	30 - 36 cm
M	47 - 53 cm	36 - 38 cm	36 - 41 cm
L	53 - 60 cm	38 - 43 cm	41 - 46 cm
XL	60 - 67 cm	43 - 48 cm	46 - 51 cm
XXL	67 - 75 cm	48 - 53 cm	51 - 56 cm
XXXL	75 - 81 cm	53 - 58 cm	56 - 61 cm

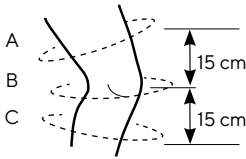

Choose the appropriate size to fit the patient, referring to the size chart. If the measurements taken do not correspond to the same size, favor the size corresponding to measurement A.



CLOSED
SHORT VERSION



OPEN
LONG VERSION

Available in short (30 cm) and long version (40 cm), in open and closed design

Open version	238801
Closed version	238802

INDICATIONS

- Functional treatment of the knee during the pre- or post-operative period, as well as the rehabilitation period (including following moderate or severe cruciate ligament and/or collateral ligament sprains).
- Conservative treatment of knee ligament injuries and/or ruptures.
- Knee instability.

The medical device mentioned on this document is class I CE marked according to the European Regulation 2017/745 on medical devices. Please contact Thuasne should you need any additional information on the device classification. Please read carefully the instructions for use, indications and contraindications of the product. TM5 patent reference: 4890607, 1990

